



cobas®

PRODUCT INF...

RELATED PRO...

Elecsys® Anti-SARS-CoV-2

Immunoassay for the qualitative detection of antibodies (incl. IgG) against SARS-CoV-2



Immunoassay to qualitatively detect antibodies (including IgG) against Severe Acute Respiratory Syndrome Coronavirus 2 (SARS-CoV-2)

Elecsys® Anti-SARS-CoV-2 is an immunoassay for the in vitro qualitative detection of antibodies (including IgG) to SARS-CoV-2 in human serum and plasma. The test is intended as an aid in the determination of the immune reaction to SARS-CoV-2.

The electrochemiluminescence immunoassay “ECLIA” is intended for use on **cobas e** immunoassay analysers. The assay uses a recombinant protein representing the nucleocapsid (N) antigen for the determination of antibodies against SARS-CoV-2.

Elecsys® Anti-SARS-CoV-2 Factsheet

Elecsys® Anti-SARS-CoV-2 Fact sheet

SARS-CoV-2: An overview of virus structure, transmission and detection

SARS-CoV-2 is an enveloped, single-stranded RNA virus of the family Coronaviridae. Coronaviruses share structural similarities and are composed of 16 nonstructural proteins and 4 structural proteins: spike, envelope, membrane, and nucleocapsid. Coronaviruses cause diseases with symptoms ranging from those of a mild common cold to more severe ones such as Coronavirus Disease 2019 (COVID-19) caused by SARS-CoV-2.^{1,2}

SARS-CoV-2 is transmitted from person-to-person primarily via respiratory droplets, while indirect transmission through contaminated surfaces is also possible.³⁻⁶ The virus accesses host cells via the angiotensin-converting enzyme 2 (ACE2), which is most abundant in the lungs.⁷⁻⁹

The incubation period for COVID-19 ranges from 2 – 14 days following exposure, with most cases showing symptoms approximately 4 – 5 days after exposure.^{3,10} The spectrum of symptomatic infection ranges from mild (fever, cough, fatigue, loss of smell, shortness of breath) to critical.^{11,12} While most symptomatic cases are not severe, severe illness occurs predominantly in adults with advanced age or underlying medical comorbidities and requires intensive care. Acute respiratory distress syndrome (ARDS) is a major complication in patients with severe disease. Critical cases are characterised, for example, by respiratory failure, shock and/or multiple organ dysfunction, or failure.^{11,13,14}

Definitive COVID-19 diagnosis entails direct SARS-CoV-2 detection by nucleic acid amplification technology (NAAT).¹⁵⁻¹⁷ Serological assays can contribute to the identification of individuals exposed to the virus and assess the extent of exposure of a population, and might thereby help to decide on application, enforcement or relaxation of containment measures.¹⁸

CONTACT US

Related Instruments

[cobas e 601 module](#) >

[cobas e 602 module](#) >

[cobas e 411 analyzer](#) >

[cobas e 801 module](#) >

Related information

[Roche's response to the COVID-19 pandemic](#) >

[Roche develops new serology test to detect COVID-19 antibodies](#) >

[Roche's cobas SARS-CoV-2 Test to detect novel coronavirus](#) >

[cobas® SARS-CoV-2 Test](#) >


[cobas® 6800 System](#) >

[cobas® 8800 System](#) >


Structure of the Severe Acute Respiratory Syndrome Coronavirus 2 (SARS-CoV-2)

- Nucleocapsid protein (N)
- Envelope protein (E)
- Spike protein (S)
- Membrane glycoprotein (M)
- RNA



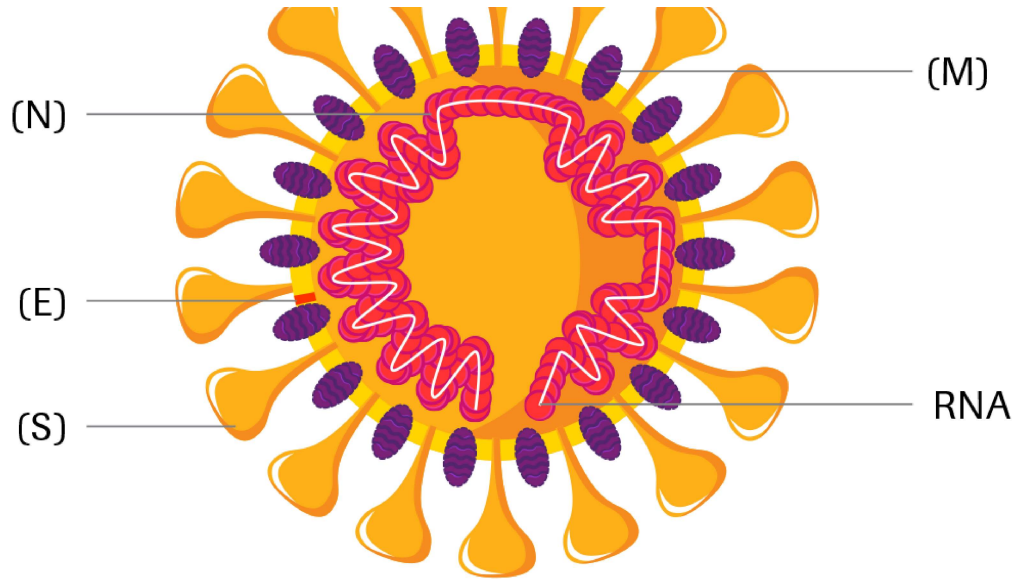

Illustrative course of markers in SARS-CoV-2 infection¹⁹⁻²⁷

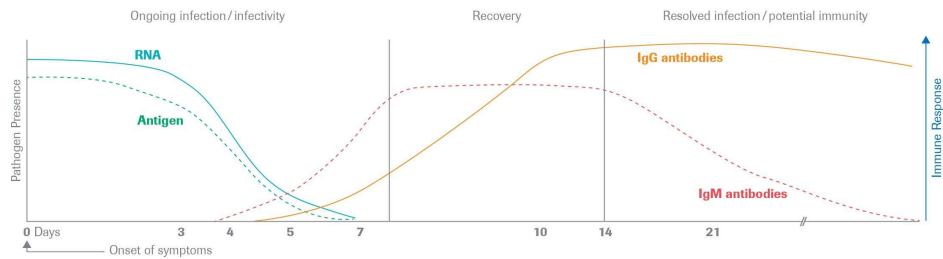

Elecsys® Anti-SARS-CoV-2

○ Systems

cobas e 411 analyzer, cobas e 601 / cobas e 602 modules, cobas e 801 module

Testing time

○ 18 minutes

Calibration

○ 2-point

Interpretation

○ COI* < 1.0 = non-reactive
COI ≥ 1.0 = reactive

○ Sample material

Serum collected using standard sampling tubes. Li-heparin, K2-EDTA and K3-EDTA plasma.

○ Sample volume

20 µL **cobas e** 411 analyzer, **cobas e** 601 / **cobas e** 602 modules
12 µL **cobas e** 801 module

Onboard stability

○ 72 hours

* COI: cutoff index

CLINICAL SPECIFICITY²⁸

[View Full Table](#)


CLINICAL SENSITIVITY²⁸

[View Full Table](#)


Screening sensitivity²⁸

SEROCONVERSION SENSITIVITY

After recovery from infection, confirmed by a negative PCR result, 26 consecutive samples from 5 individuals were tested with the Elecsys® Anti-SARS-CoV-2 assay.

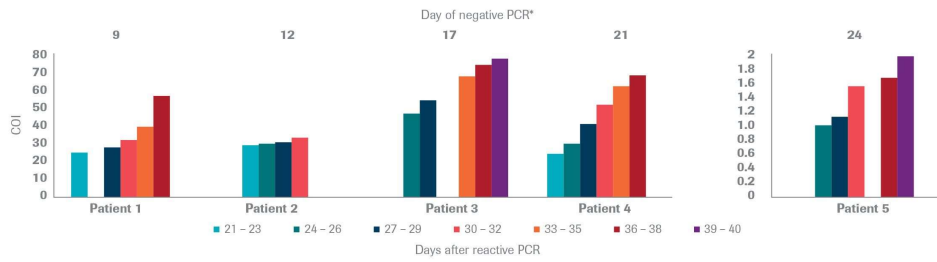


* Day 0 represents initial positive PCR

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

Roche's response to the COVID-19 pandemic


Our commitment to support India's COVID-19 testing need.

See more 

References

1. Su, S. et al. (2016). *Trends Microbiol.* 24(6), 490–502
2. Zhu, N. et al. (2020). *N Engl J Med.* 382(8), 727–733
3. Chan, J.F. et al. (2020). *Lancet.* 395, 514–523
4. CDC. (2020). https://www.cdc.gov/coronavirus/2019-ncov/prevent-getting-sick/how-covid-spreads.html?CDC_AA_refVal=https://www.cdc.gov/coronavirus/2019-ncov/prepare/transmission.html. Accessed April 15, 2020
5. WHO. (2020). <https://www.who.int/news-room/commentaries/detail/modes-of-transmission-of-virus-causing-covid-19-implications-for-ipc-precaution-recommendations>. Accessed April 15, 2020
6. Kampf, G. et al. (2020). *J Hosp Infect.* 104(3), 246–251
7. Letko, M. et al. (2020). *Nat Microbiol.*, 1–8. doi:10.1038/s41564-020-0688-y [Epub ahead of print]
8. CDC. (2020). <https://www.cdc.gov/coronavirus/2019-ncov/symptoms-testing/symptoms.html>. Accessed April 15, 2020
9. Hoffmann, M. et al. (2020). *Cell.* S0092-8674(20)30229-4. [Epub ahead of print]
10. WHO. (2020). https://www.who.int/docs/default-source/coronaviruse/situation-reports/20200403-sitrep-74-covid-19-mp.pdf?sfvrsn=4e043d03_14. Accessed April 15, 2020
11. Wang, D. et al. (2020). *JAMA.* 10.1001/jama.2020.1585
12. Huang, C. et al. (2020). *Lancet.* 10.1016/S0140-6736(20)30183-5
13. Arentz, M. et al. (2020). *JAMA.* Mar 19: e204326. doi: 10.1001/jama.2020.4326 [Epub ahead of print]
14. Wu, Z., McGoogan, J.M. *JAMA.* doi: 10.1001/jama.2020.2648 [Epub ahead of print]
15. WHO. (2020). <https://apps.who.int/iris/bitstream/handle/10665/331501/WHO-COVID-19-laboratory-2020.5-eng.pdf>. Accessed April 15, 2020

16. CDC. (2020). <https://www.cdc.gov/coronavirus/2019-ncov/hcp/clinical-criteria.html>. Accessed April 15, 2020
17. ECDC. (2020). <https://www.ecdc.europa.eu/sites/default/files/documents/Overview-rapid-test-situationfor-COVID-19-diagnosis-EU-EEA.pdf>. Accessed April 15, 2020
18. WHO. (2020). <https://www.who.int/blueprint/priority-diseases/key-action/novel-coronavirus/en/>. Accessed April 15, 2020
19. Long, Q. et al. (2020). medRxiv. preprint doi: <https://doi.org/10.1101/2020.03.18.20038018>
20. Lou, B. et al. (2020). medRxiv. preprint doi: <https://doi.org/10.1101/2020.03.23.20041707>
21. Zhao, J. et al. (2020). Clin Infect Dis. pii: ciaa344. doi: 10.1093/cid/ciaa344. [Epub ahead of print]
22. Liu, W. et al. (2020). J Clin Microbiol. pii: JCM.00461-20. doi: 10.1128/JCM.00461-20. [Epub ahead of print]
23. To ,K. et al. (2020). Lancet Infect Dis. pii: S1473-3099(20)30196-1. doi: 10.1016/S1473-3099(20)30196-1 [Epub ahead of print]
24. Xiao, D.A.T. et al. (2020). J Infect., S0163-4453(20)30138-9. doi:10.1016/j.jinf.2020.03.012. [Epub ahead of print]
25. Zhang, B. et al. (2020). medRxiv. preprint doi: <https://doi.org/10.1101/2020.03.12.20035048>
26. Wölfel, R. et al. (2020). Nature. Apr 1. doi: 10.1038/s41586-020-2196-x. [Epub ahead of print]
27. Tan, W. et al. (2020). medRxiv. preprint doi: <https://doi.org/10.1101/2020.03.24.20042382>
28. Elecsys® Anti-SARS-CoV-2. Package Insert 2020-04, V1.0; Material Numbers 09203095190 and 09203079190

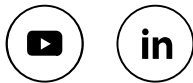

[PRIVACY POLICY](#)

[COOKIE POLICY](#)

[CONTACT US](#)

[TERMS & CONDITIONS](#)

 [INDIA / ENGLISH](#)

This website contains information on products which is targeted to a wide range of audiences and could contain product details or information otherwise not accessible or valid in your country. Please be aware that we do not take any responsibility for accessing such information which may not comply with any legal process, regulation, registration or usage in the country of your origin. Please also be aware that the information in this website should not be used to diagnose, treat, cure or prevent any disease without the advice of a qualified medical professional, and does not replace medical advice or a medical examination.

©2020 F. Hoffmann-La Roche Ltd. All trademarks used or mentioned on this Website are protected by law.

Last Updated: 12.06.2020